

Battelle CCDS™ Process

HEALTH CARE PROVIDER SIGN-UP PROCESS Battelle CCDS Critical Care Decontamination System™

BATTELLE

1 Sign up with Battelle

- Visit battelle.org/decon to fill out the enrollment form
- Battelle emails enrollee links to the enrollment contract, instructions, and the Battelle POC

battelle.org/decon



2 Contact Us to Get Your Code

- Enrollee signs contract and contacts Battelle POC to receive their 3-digit codes for each facility



3 Properly Label Respirators

- Once the 3-digit codes are received from Battelle, enrollee collects N95 respirators
- N95 respirators must be unsoiled (free of blood, mucus, make-up, lip balm, etc.) and labeled with a permanent marker



4 Collect & Bag All N95 Respirators

- Enrollee collects all N95 respirators into a single plastic bag
- Once the plastic bag is filled, tie off the bag and put it into another plastic bag



5 Properly Package

- Clean the outside bag with disinfectant
- Shipping box must be labeled with the 3-digit code and a biohazard sticker



6 Ship to CCDS Site

- Enrollee contacts their chosen logistics provider to coordinate pick-up and delivery of their N95 respirators
- Enrollee can either use a logistics provider of their choice or Battelle's preferred logistics provider



7 Decontaminated & Returned

- Your shipments are barcoded to ensure chain of custody
- Your N95 respirators are processed and then verified to ensure they are free of decontaminant
- Your decontaminated N95 respirators are returned to your facility



Battelle CCDS™ Process

HOW IT WORKS

Battelle CCDS Critical Care Decontamination System™

BATTELLE

Your Facility's Responsibility

- Your Battelle POC will provide you with 3-digit codes for each facility
- The 3-digit code must be written on each individual N95

- Collect all N95 respirators into a single plastic bag
- Once the plastic bag is filled, tie off the bag and put it into another plastic bag

- Clean the outside bag with disinfectant
- Shipping box must be labeled with the 3-digit code and a biohazard sticker

- You can either use a logistics provider of your choice or Battelle's preferred logistics provider



Battelle CCDS™ Process

- Your shipments are barcoded to ensure chain of custody
- Your N95 respirators are exposed to the vapor phase hydrogen peroxide process
- After processing, your N95 respirators are verified to ensure they are free of decontaminant

- The logistics provider returns your decontaminated N95 respirators back to your facility